



STAR
INTERNATIONAL

Worldwide Marine
and Offshore Services

Portable Oxygen Resuscitation Kit

Part # 00730002

Instructions for Use



Portable Oxygen Resuscitation Kit

Introduction

The Portable Oxygen Resuscitation Kit is assembled by Star International using independently CE Marked products under current Medical Devices regulations in Europe. The device carries a 3-year warranty.

List of Components

1 Medical Oxygen Cylinder (optional extra)	676 – 0632 - 00	00720011
2 One Meditech Medical Oxygen Regulator	677 – 0000 - 00	00710039
3 One disposable Adult BVM Resuscitator	673 – 0564 - 00	00730003
4 One VBM Hand-held Aspirator	673 – 0448 - 20	00740037
5 One pack of disposable Airways	673 – 0571 - 00	00740038
6 One Pediatric disposable Mask & Tube	673 – 9007 - 00	00740026
7 One Adult disposable Mask & Tube	673 – 9006 - 00	00740025
8 MTM Resuscitation Mask	673 – 0569 - 00	00740039
9 Instructions		00740040
10 Star International Bag with Logo	673 – 1011 - 02	00730001

Accreditations for Medical Devices included in the Star Response Kit:

BNOS Meditech Oxygen Cylinder:

- ISO 9001:2000

BNOS Meditech Oxygen Regulators:

- CE 662942
- ISO 13485:2016
- EN ISO 13485:2016

Headstar Suction Unit (supplied by BNOS Meditech):

- CE / FDA approved
- EN ISO 13485

Intersurgical Resuscitators and Consumables:

- CE 0120
- ISO9001:2000, ISO 13485

Safety Accreditations for Medical Devices included in the Portable Oxygen Resuscitation Kit



When the system is not in use do not leave pressurized.
Ensure Medical Oxygen Cylinder Valves are closed.



The device must not come into contact with any type of lubricant or grease.
Do not lubricate any component, doing so may cause ignition or explosion.



If you have any doubts regarding the condition of the device for its intended use withdraw from service immediately and return to supplier.



The device, when in use, must not be allowed near any source of ignition ie:

- (1) naked flame
- (2) persons smoking
- (3) sparks from welding or cutting devices
- (4) open electrical appliances.



Keep the device out of sight and reach of children.



Do not use any tools or wrenches on any part of this device.



Do NOT smoke in the vicinity of this device.



Use NO Oil or Grease.

Operating Instructions

Remove the product from the package and inspect for damage. If there is any damage, do not use, contact Star International.

The Portable Oxygen Resuscitation Kit has provision for one medical oxygen cylinder fitted with a pin index oxygen valve to BSEN850.

Before use of the medical oxygen regulator it is important to check all medical oxygen cylinder labelling which must be in accordance with the regulatory requirements of the Medicines and Healthcare Products Regulatory Agency.

Assembly

- 1 Fit medical oxygen cylinder into cylinder pocket of carry bag. Secure cylinder with straps.
- 2 Fit regulator to cylinder valve, ensuring seal is in place.
- 3 Connect resuscitator oxygen hose assembly or therapy mask tubing to the outlet port on the oxygen regulator if not already connected.
- 4 Open valve on medical oxygen cylinder; check that cylinder is full and that there are no leaks.
- 5) The system is now ready for use.
- 6) Turn off valve when finished.

Refer to Operation Manual, enclosed, for further instructions.

Cleaning and Maintenance

Cleaning Instructions

The unit is supplied Non Sterile



CAUTION

The Portable Resuscitation Kit is Not Autoclavable.



WARNING

Under no circumstances should the inlet and outlet connections and connectors be washed or cleaned with any soap or solution that will leave any residues.



Use No Oil or Grease



WARNING

Trigene disinfectant wipes should be used for washing the hoses and cylinders can also safely be used on all the external surfaces of the unit and accessories. Trigene Medical Surface Disinfectant Wipes are available from Star International.



WARNING

All single use items must be replaced after use.

Contact Details

For service and refills and further product information please contact:

Star International, Star House,
Turbine Business park,
Turbine road,
Birkenhead
United Kingdom

Tel: +44 (0) 1244 504 500

E.Mail: enquiries@star-international.co.uk

Website: <http://www.star-international.co.uk>



Worldwide Marine
and Offshore Services

